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PRODUCT DESCRIPTION:

The shower seat is a light construction, made of warm in contact polyurethane. The shower seat has legs of regulated height assuring stable support. It is light, easy to clean, equipped with flow openings and additional openings which allow a patient to hold the seat for better balance.

USE:

Professional supporting equipment, simplifying/ enabling the patient to shower.

CONTRAINDICATIONS:

Contraindications to a sitting position, the lack of trunk stabilization. Patients with coordination disorders, balance and intellectual disabilities should not be left without supervision.

USING (REGULATION):

The originally packed product consists of separate parts. In order to fold it, insert the legs into the tubes attached to seat, until the telescopic mechanism latches (with characteristic click sound). Remember to set all legs in the same length. Asymmetrical length setting may cause falling down. Every time after using the seat, clean it with a moist cloth. Do not use chemical detergents.

SAFETY OF USING:

1. The use of shower seat on the rough surface (of the slope more than 5%) may result in falling down and injury.
2. It is strictly forbidden to stand on the shower seat.
3. Maximum load of the shower seat – 100 kg.
4. Do not use the shower seat in case of observing any visual or detectable damages (such as frame split) because it may result in falling down.

SERVIS AND CONSERVATION:

1. Clean the shower seat frame with a moist cloth.
2. Do not use sharp cleaning tools (risk of damaging the protective cover).
3. Clean with gentle detergent.
4. Regularly clean the shower seat's mobile elements (pawl blockade mechanism).

Keeping all the elements clean is essential for preserving the product working parameters. Keeping the walking stick for a long time in a very moist room or outdoor may result in loss of esthetic and functional parameters of the product.

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Width: 51 cm

Weight: 2,95 kg

Materials: aluminum, polyurethane, rubber.

Serial number/ production date: on the product

Using time: 5 years from selling date.

Dystrybutor:

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M. Grzeškowiaka 28/2

62-035 Kórnik

Tel: 883 113 112

Tel: 783 578 128

E-mail: sklep@medbon.pl

FAX: (61) 648 26 10

www.medbon.pl